


WISPA TOILETING SLING

Hoisting from a Chair

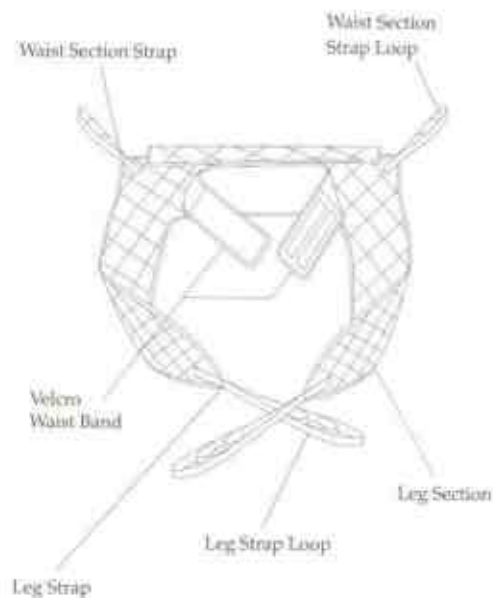

The Wispa Toileting Sling has been designed to allow quick fitment and allow good access for clothing removal and toileting.

When in use the Toileting Sling supports the client under the thighs and across the mid-back area.

Safety Note: The Toileting Sling is not considered suitable for clients who may have involuntary movement, or those clients with limited postural control.

Safety Note: A detailed assessment must be made by a qualified individual before this sling is used. Incorrect assessment may result in client discomfort or injury.

Before attempting to fit this sling ensure that the velcro fastened waistband is released.

Slide the sling down between the chair and your clients back, far enough to allow the velcro band to be fitted at waist level.



Draw the velcro band around at waist level and press together to fasten.

Comfort Note: The velcro waistband is incorporated for safety and ease of sling fitment, however it does not have to be excessively tight.



Draw each of the leg sections along and under the thigh.

Comfort Note: The quilted leg sections should be kept as flat as possible. Ensure the leg sections are positioned midway under the thighs to ensure good support and greater comfort.



Step 4

Feed one of the leg section straps through the other strap. This will keep the client's legs together during hoisting.

Comfort Note: Using the straps in this way will allow greater dignity whilst hoisting.

Comfort Note: Before fitting sling straps to the hoist, check catheter positioning, if applicable.



Step 5

Once you have completed steps 1 - 4, bring your mobile hoist or in the case of ceiling mounted hoist systems, the carry bar into position.

Safety Note: Ensure the carry bar does not approach at head level.

Attach leg and waist section straps, to the carry bar hooks.

The straps are fitted with different loop options to allow the sitting position to be altered. Please refer to the Loop Options section of this guide (page 1).



Step 6

Brakes should not be used on the hoist, this will allow hoist and chair alignment.

Comfort Note: To reduce the time your client spends hoisted, ensure the transfer area is made ready.



Step 7

Safety Note: Before commencing the lift ensure the that the sling straps are securely attached.

Complete the lift to transfer area.

THE WISPA RANGE OF SLINGS
USER GUIDE